dutchblue.com

ESCAPIES

User Manual | Evacuation mat
Gebrauchsanweisung | Evakuierungsmatte
Mode d'emploi | Tapis d'évacuation
Gebruikershandleiding | Evacuatie mat
Instrucciones de uso | Esterilla de evacuar
Istruzioni per uso | Telo portaferiti
Instrukcja użytkowania | Mata ewakuacyjna
Manual de utilização | Colchão de evacuação
Használati Utasítás | Evakuációs matrac
Priručnik za korisnike | Strunjača za evakuaciju
Ръководство за употреба | Евакуационна постелка

REF 5000

ESCAPIES

English	User Manual	4
Deutsch	Gebrauchsanweisung	5
Français	Mode d'emploi	6
Nederlands	Gebruikershandleiding	7
Español	Instrucciones de uso	8
Italiano	Istruzioni per uso	9
Polski	Instrukcja użytkowania	10
Português	Manual de Utilização	11
Magyar	Használati utasítás	12
Hrvatski	Priručnik za korisnike	13
Български	Ръководство за употреба	14

ENGLISH

ESCAPIES Evacuation mat

The ESCAPIES evacuation mat is placed between the mattress and the bed slats, allowing nursing home residents to be evacuated by a single staff member. Bed-ridden residents lying on an ESCAPIES evacuation mat can be slid across a variety of surfaces and through f.e. narrow staircases.

Instructions for use

- Remove the packaging. Do not use scissors, knives or other sharp implements when opening the packaging.
- Insert the evacuation mat between the mattress and the bed slats of the nursing bed as a standard.
- Attach the mat to the nursing bed mattress using the elastic bands at the corners.
- In the event of an emergency securely fasten the nursing home resident to the evacuation mattress using the belts.
- Use the grips at the head and/or foot of the evacuation mattress to slide the mattress (with the resident on it) across the floor into safety.

Instructions for cleaning

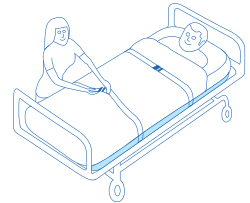
The evacuation mat can be cleaned with a soft cloth and mild detergents, for example disinfecting agent. Do not use phenol or solvents.

PLEASE NOTE

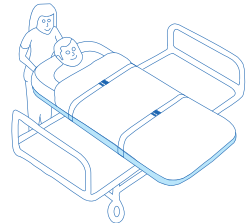
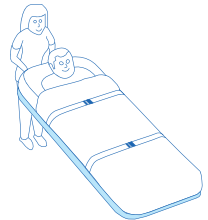
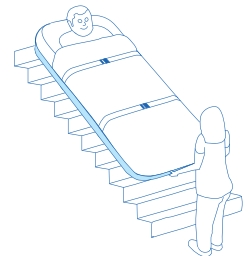
- Flame retardant according to DIN EN ISO 6940.
- Avoid contact with sharp objects.
- Load capacity of approx. 150 kg.

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Scheitlin-Papier AG