dutchblue.com

TOILET-SHOWER CHAIRIES

User Manual | Toilet-Shower Chair
Gebrauchsanweisung | Toiletten-Duschrollstuhl
Mode d'emploi | Chaise de toilette-douche
Gebbruikershandleiding | Toilet- en douchestoel
Manual de usuario | Silla de baño-ducha
Istruzioni per uso | Sedie wc e doccia
Instrukcja użytkowania | Krzesło toaletowo-prysznicowe
Instruções de uso | Cadeira de Banho e Sanitária
Инструкции за употреба | Комбиниран стол за баня и тоалетна

REF 5060

TOILET-SHOWER CHAIRIES

English	User Manual	6
Deutsch	Gebrauchsanweisung	16
Français	Mode d'emploi	26
Nederlands	Gebruikershandleiding	36
Español	Manual de usuario	46
Italiano	Istruzioni per uso	56
Polski	Instrukcja użytkowania	66
Portugal	Instruções de uso	76
Български	Инструкции за употреба	86

dutchblue makes care affordable

About Dutchblue:

We were born with a dream: We want to make care affordable for those who need it. Dutchblue strives to improve the daily work of as much people in care as possible. This will be achieved by offering a carefully selected, high quality, product range at very low prices.

How can dutchblue be so advantageous?

- We have low prices because we produce in large numbers
- We have a small product range that is carefully selected. This makes us fast and flexible, hence cheaper.
- We deliver directly to our customers. Therefore there is no one in between that will raise the price.
- We don't work with expensive external employees. This lowers the operatin expenses and thereby the prices.

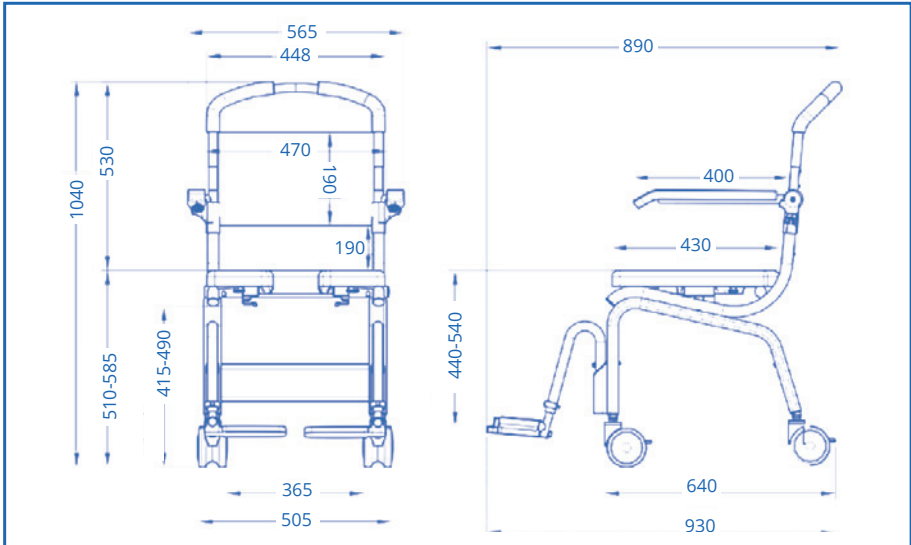
dutchblue.com

ENGLISH

Dimensions diagram:

Features	Dimensions
Height	1040 mm
Chair width - including caster wheels	505 mm
Chair width - including armrests	565 mm
Chair width - inside armrests	470 mm
Chair length - including footrest	890 mm (630 mm with footrests removed)
Backrest width	448 mm
Backrest height - from cushion to top	380 mm
Toilet height clearance (maximum)	490 mm
Seat cushion dimensions	W 460 mm x D 430 mm
Seat height from floor	510 mm to 585 mm (adjustable)
Seat cushion opening	W 175 mm x L 335 mm
Wheels	100 mm
Body weight	Max. 136 kg
Box weight	15.8 kg

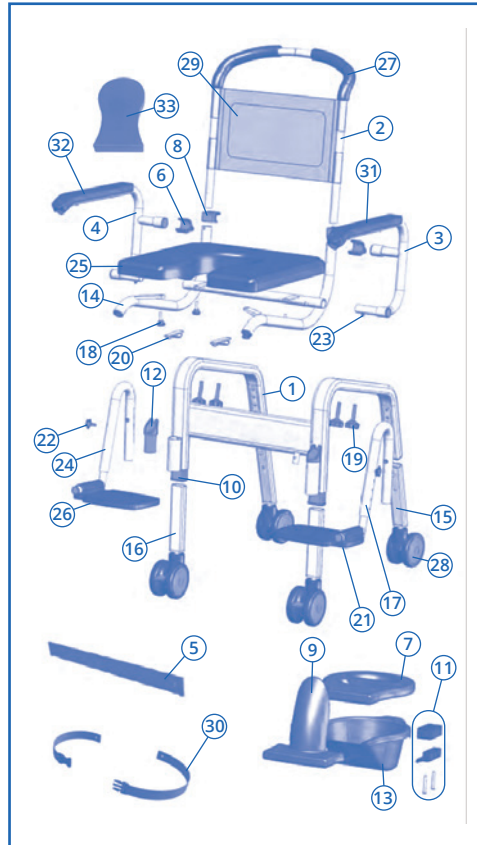
ENGLISH

Parts list:

- 1 Leg frame
- 2 Backrest frame
- 3 LH arm assembly
- 4 RH arm assembly
- 5 Safety harness
- 6 Arm lock
- 7 Commode lid
- 8 Arm lock clip
- 9 Commode deflector
- 10 Caster leg sleeve
- 11 Commode bucket connectors
- 12 Leg tube
- 13 Commode bucket
- 14 Seat frame
- 15 Rear 10 cm caster legs
- 16 Front 10 cm caster legs
- 17 Lh foot support
- 18 Seat fixing knob
- 19 Frame fixing knobs
- 20 Deflector clip
- 21 LH foot tray
- 22 Footrest adjust knob
- 23 Push pin
- 24 RH foot support
- 25 Commode cushion
- 26 RH foot tray
- 27 Foam hand grips
- 28 Rear 10 cm caster wheels
- 29 Sling seat back
- 30 Calf strap
- 31 LH arm pad
- 32 RH arm pad
- 33 Seat pad (removable)

Safety Instructions:

- DO NOT use the chair for any application other than the manufacturer's intended use as this can result in damage to the chair and its components. Not complying with the manufacturer's instructions can result in injury to the user(s) or the caregiver. Dutchblue.com LTD will not be held responsible for any injury or damage resulting from the misuse or inappropriate operation of the chair.
- ALWAYS ensure that your chair is capable of manoeuvring over surfaces that will be encountered, prior to operating the chair with any user(s). This chair is only intended for use indoors, on flat surfaces, without thresholds, steps, curbs or obstacles.
- ALWAYS lock the castor wheels when the chair is stationary. To do this step on the lever which activates the wheel locks.

ENGLISH

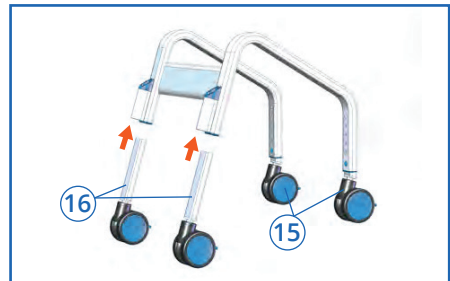
Safety Instructions:

- DO NOT stand or step on the footrest, as the chair will tip forward possibly causing injury to the User. The footrests cannot bear heavy loads and are not intended or designed as a step.
- DO NOT lean forward or sideways out of the chair as this may cause the chair to tip.
- DO NOT wheel the chair down an incline without a caregiver who can control the chair during its descent.
- DO NOT use the chair castor wheel locks as brakes when travelling down inclines.
- DO NOT position the chair sideways on a slope as this could cause it to tip.
- Never use the foot rest to assist with transfers. This could cause the chair to tip forward. The footrest should be folded away during transfers to provide unobstructed access to the chair.
- DO NOT stand on the footrest as it is designed to provide support for the user's feet while seated and not to act as a load bearing component when the user mounts or alights from the chair. Standing on the footrests could cause the chair to tip forward.
- The fasteners – knobs, nuts, bolts, washers – should be checked periodically to ensure that they are securely fastened.

Instructions for assembly:

1. Insert caster legs into leg frame. Ensure all push pins are set at the same height location to ensure the wheeled base is leveled and does not rock when sitting on all 4 caster wheels.

Note: Caster legs differ between front & rear.
Ensure the correct ones are used.

2. Put the seat frame on top of the leg frame and secure in position by screwing in the 4 fixing knobs.

Note: This part may come pre-assembled from the factory.

ENGLISH

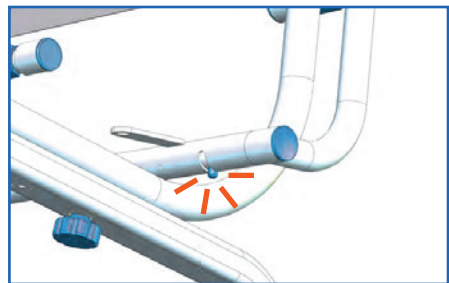
- Slide the backrest frame down through the arm clips and into the seat frame. Make sure the push pins are fully depressed before inserting the backrest frame.

- Fit the left hand arm and the right hand arm by positioning the axle (smaller diameter) part of the tube against the hollow tube across the rear of the seat frame and sliding into this tube. The push pin will need to be pushed in (depressed) to allow the arms to be fully inserted into this tube.


Once fully inserted the push pin will “pop out” into the formed slot to lock the arm into position. This prevents the arm from being removed again without depressing the push pin.

- Fit the commode cushion by positioning onto the seat frame and align the 4 fixing locations. Fix down using the 4 seat fixing knobs and tighten. The cushion will have the commode bucket connectors pre-fitted at the factory.

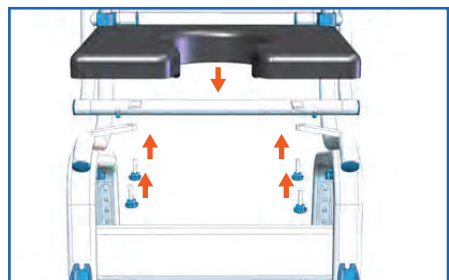
Note: this part may come pre-assembled from the factory.


ENGLISH

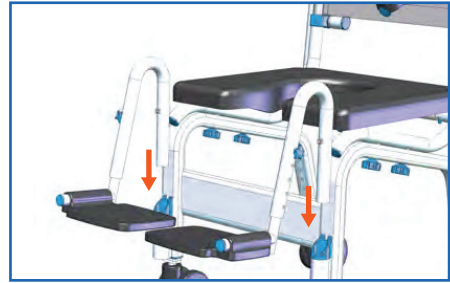6. To install these assemblies insert the Left and right hand footrest support Tubes into the seat frame as shown on the diagram. Ensure the location pins are seated inside the location groove shown.
Note: The footrest assemblies will come pre-assembled from the factory.

Adjust the footrest height to suit the User as shown ensuring that the screw Lock is fully tightened and the foot supports are level.

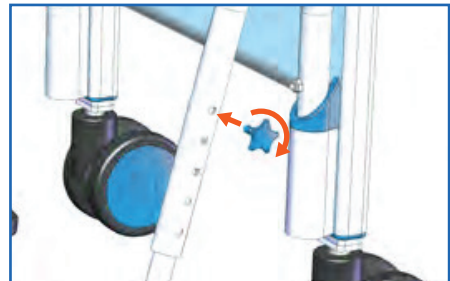Note: The distance between the footrest and the seat cushion can be increased further, if required, by raising the seat height.

To set the seat height adjust the Caster legs by using the push pins in The legs and ensure that the push pins Fully engage the height adjustment Holes evenly.

7. The Commode Bucket can be pushed into position from the rear of the chair and is held in position by the Commode Bucket Connectors pre-fitted to the cushion at the factory. The Commode Bucket lid is for use after the bucket has been removed from the chair.

ENGLISH

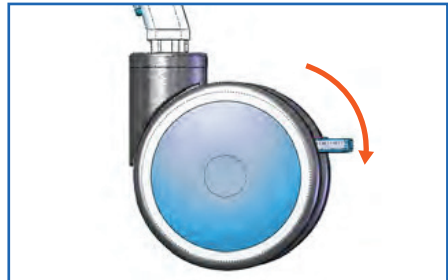
A commode deflector is able to be fitted to the commode bucket after the bucket is fitted to the chair. The deflector may be fitted after the user is seated to allow easier transfers of the user to and from the chair. You can also fit the deflector without the bucket by sliding it into the factory fitted clips on the seat base. The chair is now fully assembled and ready for use.


Operation:

Caster Wheels

All caster wheels can be locked and unlocked by depressing the foot brake pedal on each caster wheel. All caster wheels should be locked during the transfer of the user to and from the chair.

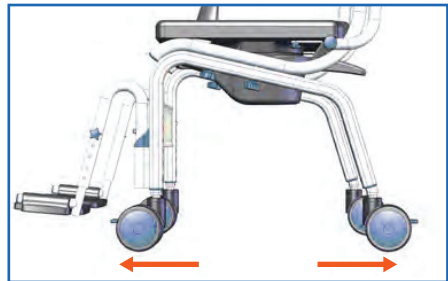
The caster wheels should be rotated outwards towards the front and rear of the chair to make the chair more stable during transfers.

SEAT HEIGHT ADJUSTMENT

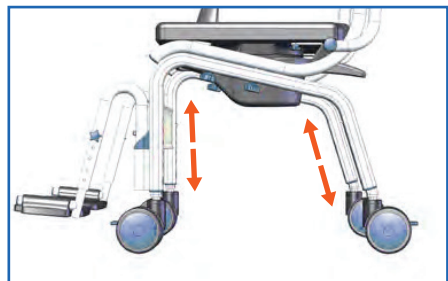The chair may be adjusted to have a higher seat height at the front and lower at the back to suit user requirements. This can be accommodated by adjusting the front caster leg only to be extended further than the rear caster leg. Under no circumstances should there be more than 1-adjustment position difference between front and rear (approximately 20mm).

ENGLISH

FOOTREST HEIGHT ADJUSTMENT AND FOLDING

The tray footrests fitted to your chair are not designed to bear user weight. They are designed for the user to rest their feet.

WARNING - DO NOT STAND on the footrest as the chair may become unstable and tip forward causing injury to the user.

To raise or lower the footrests unscrew the locking knob in the footrest supports and move the footrest supports up or down to suit the user height requirement.

If the user is very tall and require additional seat cushion to footrest adjustment, raise the seat height (as explained earlier) while having the footrest support at the lowest setting. The footrests can be folded and swung away when not in use or during transfers.

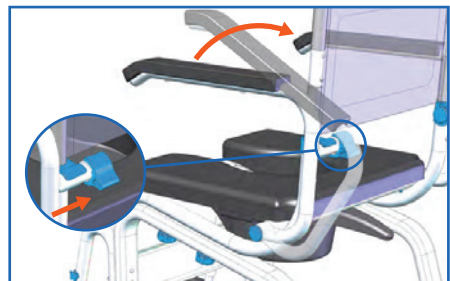
1. Lift the footrest plate and fold it up until required to be lowered again.
2. Lift footrest assembly and rotate to swing the entire leg out and away if more space is required.

To completely remove the footrest simply lift away from the main seat assembly.

ARMRESTS LOCKING AND RELEASE MECHANISM

You can easily swing the armrest up and out of the way or secure the armrest in downward position. To unlock the armrest slide the arm lock towards the centre line of the chair back as shown, to allow the arm to be moved upwards and out of the locked position.


ENGLISH


To lock the armrests just push the armrest back down into the lowered position and slide the lock back to its original position. Ensure the slide lock is fully over the plastic clip.

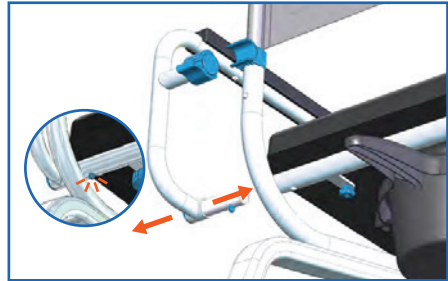To fully remove an armrest from the chair, depress the push pin fully allowing the armrest to be pulled outwards from the seat frame tube. To re-fit the arm, reverse this action.

SEAT CUSHION

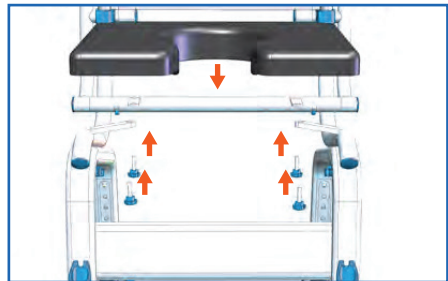The seat cushion can be removed for cleaning or to change from open front to one of the accessory options with open side or closed front by releasing the seat fixing knobs underneath the seat cushion.

COMMODOE BUCKET AND COMMODOE DEFLECTOR

Connectors hold the commode bucket to the chair and are factory assembled to the underside of the cushion. Slide the commode bucket in from the rear of the chair and locate the bucket into the slide/connectors. Push the bucket in until it stops against the connector end.

A commode deflector can be fitted to the front of the commode bucket if required, after the bucket has been fitted to the chair, allowing the user to transfer easily to the chair prior to fitting the splash guard. The commode cover should be used when transferring contents to the toilet to avoid spills.

ENGLISH

SAFETY RESTRAINTS

A safety restraint is supplied with the chair and should be used. It can also be used as a waist restraint if preferred. You can also purchase an additional harness if you require waist and chest restraints.

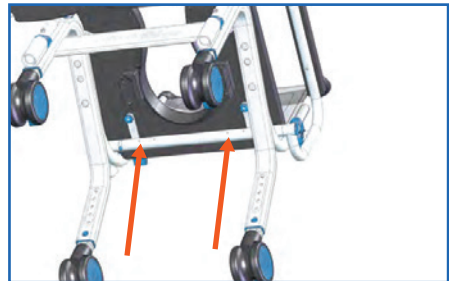
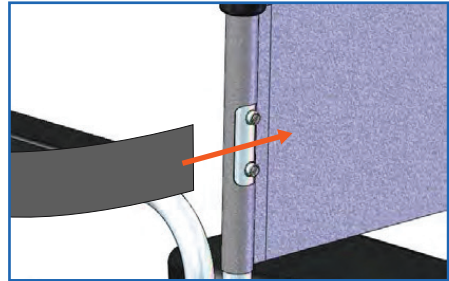
To fit or remove the restraints unscrew the clamp and remove the belt. Make sure the clamp is tight enough to prevent the belt pulling through when refitting.

Clamp bars are fitted to the chair back and at the lower lap area on the seat frame.

Before use, adjust the length of the safety restraints to suit the user.

Fit your harness to the holes as shown if you prefer to use it as a waist restraint.


Fitting two harnesses adds extra support if necessary.

LEG RESTRAINT

An additional calf strap is supplied with your chair. We recommend that the supplied harnesses are installed and used at all times.

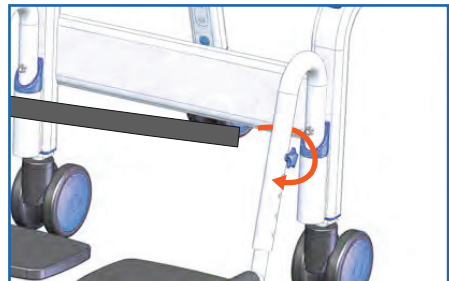To fit, simply wrap your harness around the footrest support arm as shown, making sure that the harness is above the locking knob, this will prevent it from slipping down during use.

ENGLISH

Cleaning & Maintenance:

1. Regularly inspect all parts of the chair for defects. Any defective part should be replaced promptly.
2. Clean the seat using a damp cloth and a mild cleanser. Always read the manufacturer's instructions prior to applying the cleanser. When using sodium hypochlorite (bleach) on the seat, a solution of 0.1% sodium hypochlorite should have no adverse effects on the cleaning of the chair. However, when using a solution of 1.0%, rinse the chair thoroughly with clean water after application. Over a period of time bleaching may occur, but the integrity of the product will remain unaffected.
3. Dry off the chair after use.
4. Do not oil or grease any parts.
5. Keep the castors free from lime scale, soap residue and hair by cleaning after use in the shower.

Stainless steel has excellent corrosion resistance, but it is not stain- or rust-proof. Regular maintenance will keep stainless steel clean and free of contaminants. In most applications stainless steel will not rust or stain even after many years of service, but when used in certain environments such as concentrated chlorine or other corrosive elements stainless steel requires proper care and maintenance. Do not leave the stainless steel in contact with iron, steel or other metals.

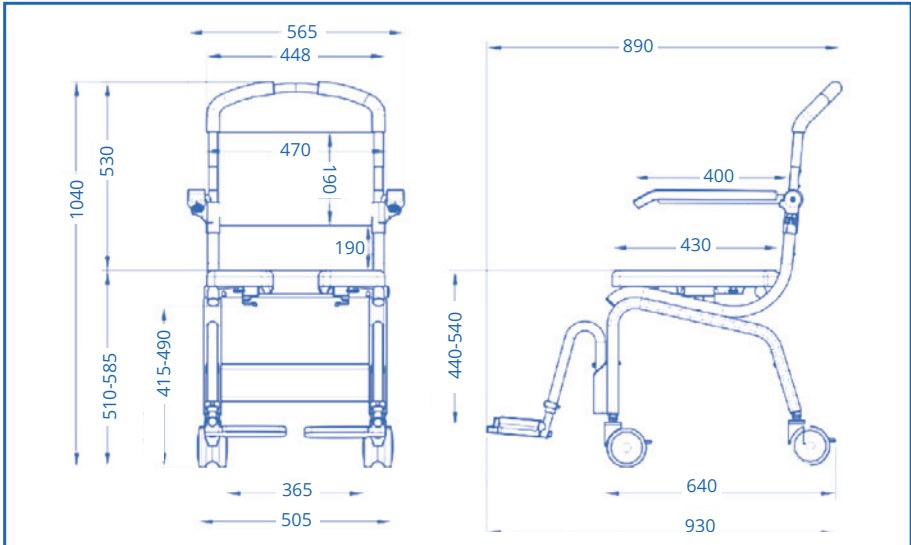
- Clean stainless steel frequently with soap and water. Any detergent which is safe for glass is usually safe for stainless steel, too. Apply it with a cotton cloth.
- Use a coating such as a good wax to help safeguard the stainless steel.
- Remove discolouration or rust spots as soon as possible with a brass, silver or chrome cleaner. Irreversible pitting will develop under rust that remains on stainless steel. Apply a mild cleaner with a wet cotton cloth. Rinse thoroughly and wipe dry.
- DO NOT use abrasives such as sandpaper or steel wool on the stainless steel.
- DO NOT clean with mineral acids or bleaches.
- DO NOT store near any chlorine product.

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Scheitlin-Papier AG