dutchblue.com

BATH CHAIRS WITH BACKREST

User Manual | Bath chair with backrest
Gebrauchsanweisung | Duschhocker mit Rückenlehne
Mode d'emploi | Chaise de douche avec dossier
Gebruikershandleiding | Douchekruk met rugsteun
Manual de usuario | Silla de baño con respaldo
Istruzioni per uso | Sgabello per doccia con schienale
Instrukcja użytkowania | Krzesło kąpielowe z oparciem
Manual do usuário | Cadeira de banho com costas
Ръководство за употреба | Стол за баня с облегалка

REF 7507

BATH CHAIRS WITH BACKREST

English	User Manual	3
Deutsch	Gebrauchsanweisung	4
Français	Mode d'emploi	5
Nederlands	Gebruikershandleiding	6
Español	Manual de usuario	7
Italiano	Istruzioni per uso	8
Polskie	Instrukcja użytkowania	9
Português	Manual do usuário	10
Български	Ръководство за употреба	11

ENGLISH

Bath chair with backrest

Water-permeable bath chair with backrest, anti-slip seat and non-slip knobs on the feet.

Material: Powder-coated aluminum frame. 5-way height adjustable seat. Maximum user weight 100 kg. Ideal for every shower.


Composition

Item	Description	Model	Quantity
1	Upper tube		1
2	Bottom tube		1
3	Telescopic tube		4
4	Screw	M6x55	4
5	Vaulted washer	Ø6	9
6	Screw	M6x35	1


Item	Description	Model	Quantity
7	Backrest tube		1
8	Screw	M6x30	4
9	Backrest board		1
10	Seat board		1
11	Spring pin	Ø8	4

Instructions for assembly


1. Assemble together the top and bottom tubes according to image 1.
2. Place the seat downward horizontally and place the two fixed tubes into the groove, making sure that marking 1 corresponds to marking 1 and marking 2 corresponds to marking 2 as shown in image 2. Place the assembled backrest onto the two fixed foot tubes. Insert and tighten all screws.
(note: fix the long part of the backrest tube with the backrest and the short part - with the seat)
3. Place the chair properly and adjust equally the telescopic tubes to suitable length, making sure the spring pin is in place before sitting on.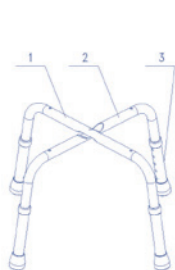

①

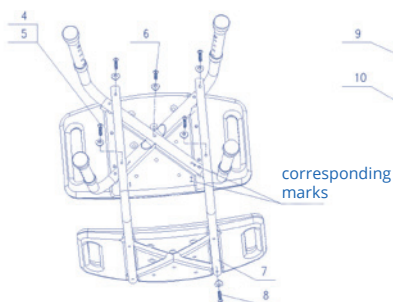②

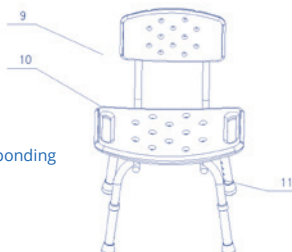③

Safety and maintenance

Before use, check the degree of tightness of all screws and stability of the frame, don't overload above 100 kg. Clean it frequently and disinfect it in periodically. Place it in dry and ventilated environment.

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Scheitlin-Papier AG