dutchblue.com

HEIGHT-ADJUSTABLE RAISED TOILET SEATIES

- User Manual** | Height-Adjustable Raised Toilet Seat with armrests
Gebrauchsanweisung | Höhenverstellbare Toilettensitzerhöhung mit Armlehnen
Mode d'emploi | Siège de toilette surélevé réglable avec accoudoirs
Gebruikershandleiding | Toileetverhoger met armleuningen, hoogte verstelbaar
Manual de usuario | Asiento de inodoro elevado y ajustable en altura con apoyabrazos
Istruzioni per uso | Sedile WC rialzato regolabile in altezza con braccioli
Instrukcja użytkowania | Nasadka podwyższająca, z podłokietnikami, regulacją wysokości
Instruções de uso | Assento sanitário elevado ajustável em altura com apoio de braços
Инструкции за употреба | Повдигната седалка за тоалетна с подлакътници и регулируема височина

REF 7509

dutchblue.com

HEIGHT-ADJUSTABLE RAISED TOILET SEATIES

English	User Manual	4
Deutsch	Gebrauchsanweisung	8
Français	Mode d'emploi	12
Nederlands	Gebruikershandleiding	16
Español	Manual de usuario	20
Italiano	Istruzioni per uso	24
Polskie	Instrukcja użytkowania	28
Portugal	Instruções de uso	32
Български	Инструкции за употреба	36

ENGLISH



Set-up and operation

Designed to assist with toilet transfers, this adjustable toilet seat raiser attaches to existing toilet seat mounting points. Can be used with or without armrests (one or both). Seat height can be adjusted to 50 mm, 100 mm or 150 mm.

- ⚠ Prior to use please read all instructions. Unit must be properly mounted to an existing toilet bowl before use.

Installation

1. Remove your existing toilet seat from the bowl. Seats are generally attached by a pair of bolts/ nuts located under the rear rim of the toilet.
2. Unscrew and remove the long plastic nuts from the rear of the toilet seat raiser.
3. Position the unit onto your toilet bowl, inserting the rear mounting bolts through the existing holes on your toilet. Ensure the unit is centred over the toilet bowl.
4. The position of the mounting bolts can be adjusted to match different toilets. To adjust, slide the mounting bolts along the rear bar to achieve the correct distance.
5. Mounting nuts must be loose for mounting bolts to slide freely along the bar. Reattach both long plastic nuts to the mounting bolts and tighten firmly. Check the toilet seat raiser is firmly clamped against the toilet bowl before use.

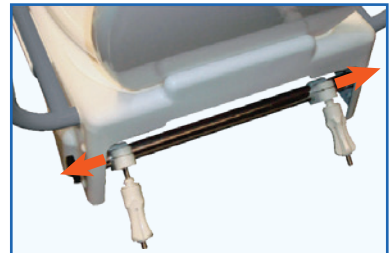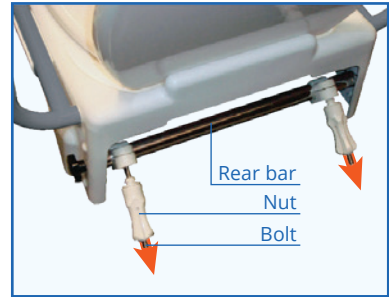ENGLISH

Adjusting the seat height

The toilet seat raiser can be adjusted to 3 different height settings, adding 50, 100 or 150 mm to the height of the toilet bowl. Height can be adjusted either before or after installation. When adjusting after the toilet seat raiser is installed, be careful not to drop components into the toilet.

Ideal seat height

Ideal height is when the user's feet can completely touch the floor, and the hip height is higher than knee height (ie less than 90 degrees on the knee joint or hip joint).

Seat height is adjusted by setting both front and rear bars to 1 of 3 positions, as dictated by a set of 3 height adjustment holes. For 50 mm height, set bars to top hole. For 100 mm height, set bars to middle hole. For 150 mm height, set bars to bottom hole.

1. To reposition the bars, remove the bolt protector caps from each side of the toilet seat raiser (front and rear).
2. Flip up the toilet seat for easy access to the bars. Using the inbuilt adjustable knob, undo the front bolt and remove. Repeat on the opposite side, holding the bar to prevent it falling.
3. Reposition the bar to the desired height. Holding the bar in place, reinsert the adjustment knob on both sides and tighten until firm.
4. The rear bar is adjusted in the same way, using the inbuilt adjustment knob to remove side bolts. This adjustment can be used to adjust the rear height setting to facilitate a forward angled tilt for users requiring extra assistance when standing (rear bar lower than front bar).
5. When adjustment is complete, replace all bolt protector caps.

⚠ Do not position the front bar higher than the rear bar, as this will create a rearward tilt that may cause injury to the user.

ENGLISH

Inserting armrests

The toilet seat raiser can be used with or without armrests. One or both armrests can be used to suit individual users and/or bathroom set up. A brass locking button is used to secure each armrest.

To insert the armrests, press in the locking button located under the frame of the armrest and slide the armrest into the arm hole at the side of the toilet seat. Check the armrest is stable and secure before use.

Using the Toilet Seat Raiser

- The toilet seat raiser operates just like a standard toilet seat. The seat may be flipped up so the bowl can be accessed without the seat.
- Armrests may be swung up and out of the way for increased access. Pull gently upwards to unclip the armrest, and swing upwards until vertical.
- Armrests may also be removed completely for ease of transfer or cleaning. Swing armrests up and flip up the toilet seat to access the armrest locking button. Press the locking button in and pull the armrests out. Store safely when not in use.

ENGLISH

Regular maintenance

- Regularly check the mounting screws are tight and that the toilet seat raiser is firmly clamped to the toilet bowl.
- Regularly check the toilet seat raiser for damage and wear, paying special attention to the condition of the seat and the mounting attachment. Do not use the toilet seat raiser if the plastic seat is cracked or deformed in any way or if you see evidence of any other damage or general wear.

Care & cleaning

- To clean the unit after each use, wipe with a mild soap and warm water solution and rinse well. Dry with a clean cloth.
- For heavier soiling, wipe clean with a diluted bleach solution and rinse well before drying.
- Do not use water temperatures exceeding 80 °C.
- Do not use high pressure cleaners, steam cleaners or harsh, abrasive cleaning agents or utensils (such as wire brush) as these can lead to deterioration of the unit.

Specifications

Max. weight capacity	136 kg
Adjustable heights	150 mm
	100 mm
	50 mm
Product weight	3.8 kg (net)

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Scheitlin-Papier AG